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15-30°C

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Testing instructions - English

15-30°C

Read the instructions, wash your hands, collect the contents and bring the test to room temperature.

Note! The front of the package tells you which test to perform. Check carefully which instructions to follow.

Tip!

Take a picture of your code card for the results.

A

Vaginal

Female

1

2

3

4

CODE
012345678

B

Urine

Male

1

2

3

4

CODE
012345678

2h

C

Oral

1

2

3

4

CODE
012345678

D

Anal

1

2

3

4

CODE
012345678

A - Vaginal

- 1 - Remove the code card from the return box and keep this code well.
- 2 - Spread the labia hold the cotton swab. Insert the cotton swab 5 cm into the vagina. Stroke along the vaginal wall for 30 seconds.
- 3 - After you are done put the cotton swab back into the tube.
- 4 - Close the tube tightly. Put it in the Safety Bag and tape it tightly shut. Put the Safety Bag in the return box and fold/tape it closed.

B - Urine

1 - Remove the code card from the return box and keep this code well.

2 - Preferably take morning urine, or wait at least two hours after your last urination before starting the collection. Take a clean cup (e.g. disposable cup) and collect the first drops/jet of your urine (max. 30 ml).

3 - Open the tube. Use the pipette to introduce 1.5 ml of urine into the tube.

4 - Turn the tube tightly closed. Put it in the Safety Bag and tape it closed. Put the Safety Bag in the return box and fold/tape it closed.

C - Oral

1 - Remove the code card from the return box and keep this code carefully.

2 - Remove the cotton swab from the package. Make sure it is not in contact with anything. Hold the cotton swab on the break line.

3 - Insert the cotton swab into the back of your throat and gently rotate it and rub firmly along the left and right palate arch. Note: This may cause a gagging sensation. If this happens, take a break and then continue until you have rubbed at least 6 times.

4 - Insert the cotton swab into the tube. Now twist the tube tightly. Put it in the Safety Bag and tape/fold it closed.

D - Anal

1 - Remove the code card from the return box and keep this code carefully.

2 - Remove the cotton swab from the package. Make sure it is not in contact with anything. Hold the cotton swab on the break line.

3 - Carefully insert the cotton swab 3-5 cm into the anus and rotate it for 5-10 seconds.

4 - Place the cotton swab in the test tube without touching other surfaces. Now twist the tube tightly. Put it in the Safety Bag and tape/fold it closed.

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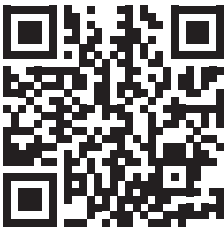
Having problems with the manual? Read the instructions only by scanning the QR-code or go to <https://instructie.thuistest.shop>

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EN - After you have sealed the Safety Bag, carefully put it in the return box. Send the return box to the laboratory by putting it in an orange PostNL street mailbox before 17:00 on a weekday.

Is it colder than 0 degrees or warmer than 30 degrees outside? Then drop off the shipping envelope at a PostNL point. 1-2 working days after receipt of your sample at the laboratory, your results can be viewed at uitslag.thuisest.shop. Enter your sample number and receive your results anonymously.

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EN - In Belgium, you send your package via a mailbox from Bpost.

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This user manual is provided for informational purposes only and is intended to guide the proper use of the STD testing kit. While we strive to ensure that the information contained within this manual is accurate and up to date, we do not guarantee its completeness, reliability, or accuracy. The information provided may be incomplete, incorrect or subject to change without notice.

Thuistest.shop shall not be held responsible for any errors, omissions, or misuse of the information provided in this manual. Additionally, we disclaim any liability for damages, injuries, or adverse consequences resulting from reliance on the information contained herein or the improper use of the testing kit.

By using this product, you acknowledge and accept that thuisest.shop is not liable for any outcomes arising from its use, including but not limited to misinterpretation of results or failure to seek appropriate medical care.

If you have any questions or concerns regarding the use of this test, please contact a healthcare professional or contact [thuisest.shop](mailto:info@thuisest.shop) via info@thuisest.shop.

©thuisest.shop all rights reserved, typographical and printing errors reserved