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# User instructions STD-test (anal)

With this test-set, you can test yourself for chlamydia & gonorrhoea (anal). Check before testing if the kit is complete.

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1. Take the cap off the test tube.
2. Pick up the Q-tip by holding it with your thumb and index finger. Avoid contact with any other surfaces.
3. Place the Q-tip in the back of your throat and turn it carefully
4. Place the Q-tip 3-5 cm inside the anus and turn it carefully for 10 seconds.
5. Place the Q-tip in the test tube without touching any other surfaces. Proceed to close the tube firmly.
6. Place the tube, following the instructions, in the DaklaPack Safety Bag. Remove the white stroke from the Safety Bag and close it firmly.
7. Note: take the card with the sample number out of the bag and keep it safe. You need the sample code to get your results on [uitslag.thuistest.shop](https://uitslag.thuistest.shop).
8. Put the Safety Bag in the (cardboard) return box. You can close it more firmly by using the sticker that is provided.
9. Send the return box to the laboratory by mailing it before 17.00 on a weekday in one of the orange PostNL mailboxes. You do not need any post stamps. Is it colder than 0 degrees or hotter than 30 degrees outside? Then drop off the package at a PostNL drop-off point.
10. 1-2 working days after receiving the sample in the laboratory, your results will be available on [uitslag.thuistest.shop](https://uitslag.thuistest.shop). Fill in your sample number and get your results anonymously.

## This kit contains:

- 1x Test tube
- 1x Q-tip
- 1x A white (cardboard) return box from Sÿsse with an answer number sticker
- 1x Result card with sample number
- 1x DaklaPack Safety Bag
- 1x Closing sticker for the box

**Tip:** Take a picture of your code card and sample number so you won't lose them when getting the results!