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# User instructions STD-test man (urine)

With this test-set you can test yourself for chlamydia & gonorrhoea. Check before testing if the kit is complete.

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1. Take a clean cup and catch the first droplets/stream of urine in it (max. 30 ml). You only need a small bit of urine. Preferably use morning urine.
2. Take the cap off the test tube.
3. Aspirate some of the urine with the pipette and place it in the test tube. If there is 1.5 ml of urine in the test tube it is enough.
4. Then screw the tube tightly.
5. Place the tube in the DaklaPack Safety Bag according to the instructions. Remove the white strip from the Safety Bag and tape it tightly.
6. Note: Remove the storage card with your sample number, and keep it in a safe place. You will need the sample code to request the results via [uitslag.thuigestest.shop](https://uitslag.thuigestest.shop)
7. Put the Safety Bag in the (cardboard) return box.
8. Send the return box to the laboratory. This can be done before 17.00 on a weekday in an orange PostNL street mailbox. You do not need to put post stamps on. Is it colder than 0 degrees or hotter than 30 degrees outside? Then drop off the shipping envelope at a PostNL point.
9. 1-2 working days after receiving the sample in the laboratory, your result can be seen via [uitslag.thuigestest.shop](https://uitslag.thuigestest.shop).

## This kit contains:

- 1x Test (urine) tube
- 1x Absorption bag
- 1x A pipette wrapped in plastic
- 1x A white (cardboard) return box from Sÿsse with an answer number sticker
- 1x Result card with sample number
- 1x DaklaPack Safety Bag
- 1x Closing sticker for the box

**Tip:** Take a picture of your code card and sample number so you won't lose them when getting the results!