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User instructions STD-test (oral)

With this test-set you can test yourself for chlamydia & gonorrhoea (oral). Check before testing if the kit is complete.

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1. Take the cap off the test tube and place it in the tube holder.
2. Pick up the Q-tip by holding it on the break line using your thumb and index finger. Avoid contact with other surfaces.
3. Place the Q-tip 3–5 centimeters in the throat and turn it carefully for 10 seconds.
4. Place the Q-tip in the test tube without touching any other surfaces. Proceed to close the tube firmly.
5. Place the tube, following the instructions, in the DaklaPack Safety Bag. Remove the white stroke from the Safety Bag and close it firmly.
6. Note: Take the card with the sample number out of the bag, and keep it safe. You need the sample code to get your results.
7. Put the Safety Bag in the (cardboard) return box. You can close it more firmly by using the sticker that is provided.
8. Send the return box to the laboratory by mailing it before 17.00 on a week day in one of the orange PostNL mailboxes. You do not need any post stamps. Is it colder than 0 degrees or hotter than 30 degrees outside? Then drop off the package at a PostNL drop off point.
9. 1-2 working days after receiving the sample in the laboratory, your results will be available on uitslag.thuistest.shop. Fill in your sample number and get your results anonymously.

Deze kit contains:

- 1x Test tube
- 1x Q-tip
- 1x A white (cardboard) return box from Sÿsse with an answer number sticker
- 1x Result card with sample number
- 1x DaklaPack Safety Bag
- 1x Closing sticker for the box

Tip: take a picture of your code card and sample number so you won't lose them when getting the results!