8-in-1 urine test

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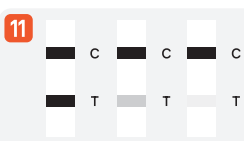
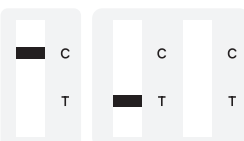
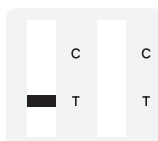
Ongeldig

8-in-1 urine test

Before you begin this test, read the instructions carefully, ensure that all the necessary items for the test are available, and wash your hands before beginning the test.

If any items for the test are missing, take a new test to ensure an accurate result.

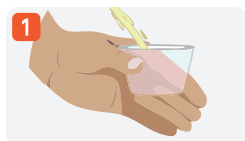
Scan QR-code to see video instruction


Contents of this kit


- Sample buffer tube
- Transparent plastic cup
- Yellow plastic cup
- Pipette
- Test cassette
- Color scale reference chart
- Package insert

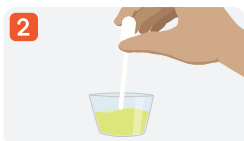Take out the transparent plastic cup and use it to collect your urine.

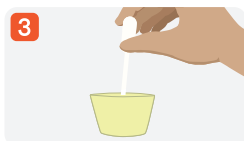Pipette 5 ml of the collected urine from the urine catcher.

Add the 5 ml of the collected urine into the yellow plastic cup.

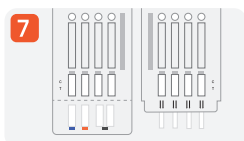
Carefully open the buffer tube and make sure not to spill.


Pour the buffer liquid into the yellow plastic cup.


Use the dropper to mix the urine and buffer liquid in a similar clockwise direction for a total of 10 times.

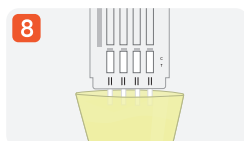
Take out the test cassette from the foil pouch. Open the bottom of the cassette cover.

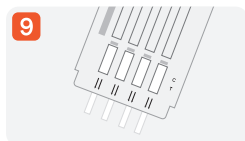
Place the cassette into the yellow plastic cup and observe to make sure that each test strip is immersed in the liquid. Do not exceed the height indicated by the arrow!

Keep the test device flat on the table.

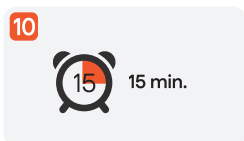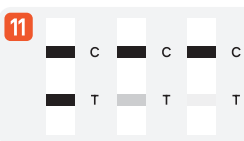Read the results after 15 minutes. Interpret the results using the reference chart provided in the test kit. Tip: set a timer on your phone.

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