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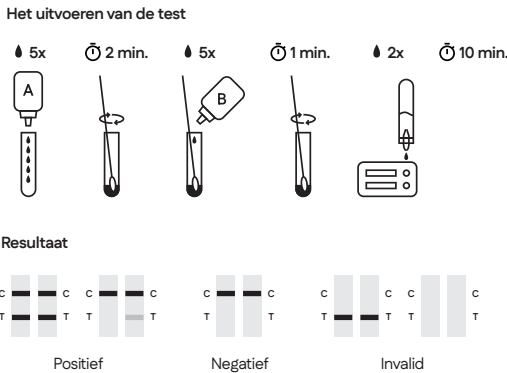
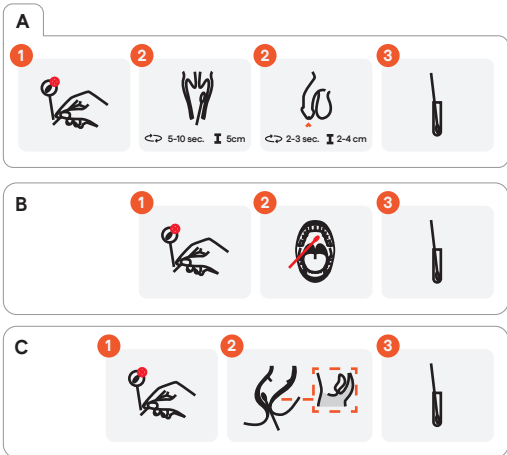
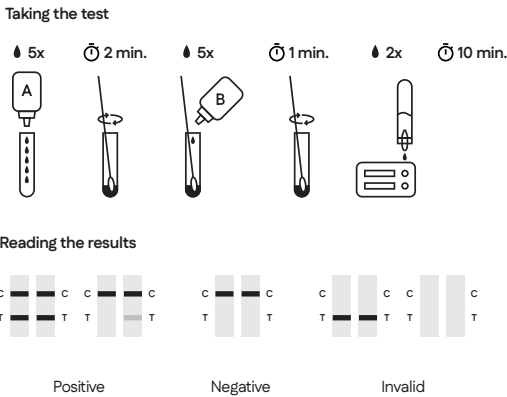
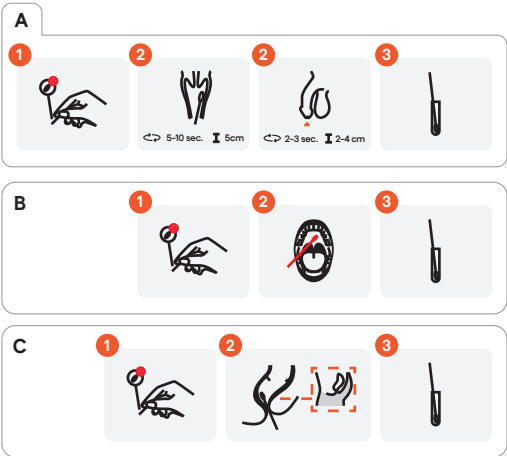
Testing instructions - English

Read the instructions, wash your hands, collect the contents and bring the test to room temperature.

A - Taking the test - vagina/penis (urethral)

For men: do the test preferably in the morning before you have not been to the toilet for at least 2 hours.

1 - Open the package: remove the cotton swab from the package without touching the tip.

2 - (Women) Insert the cotton swab: sit or stand in a comfortable position. Carefully insert the cotton swab 5 cm into the vagina and rotate it for 5-10 seconds. Make sure the cotton swab touches the vaginal walls.

2 - (Men) Insert the cotton swab: Insert the cotton swab about 2-4 cm into the urethra and twist for 2-3 seconds. This may be sensitive.

3 - Remove the swab and insert it into the extraction tube

B - Taking the test - oral

1 - Open the package: remove the swab without touching the tip.

2 - Insert the swab: Insert the cotton swab into the back of your throat and gently rotate it and rub firmly along the left and right palate arch. Note: This may cause a gagging sensation. If this happens, take a break and then continue until you have rubbed at least 6 times.

3 - Remove the swab and place it in the extraction tube

C - Taking the test - anal

1 - Open the package: remove the swab from the package without touching the tip.

2 - Insert the swab: Insert the swab 3-5 cm into the anus and gently rotate it for 10 seconds

3 - Remove the swab and place it in the extraction tube

Perform test

1 - Put 5 drops of bottle A into the extraction tube: many more drops are not necessary, more liquid will come into the tube later;

2 - Add the cotton swab with the extraction and swirl for 2 minutes and at least 15 times: handy to turn on a timer on your phone to keep track of time;

3 - Put 5 or 6 drops of bottle B into the extraction tube: the liquid may now start to get a bit cloudy;

4 - Then swirl 10 more times or for at least 1 minute: usually the solution becomes clear with a slight green or blue tint. If the swab was a little dirty or bloody, the color is going to turn yellow or brown. This is normal.

5 - Take out cotton swab, leaving as much liquid as possible in the tube: Press the cotton swab against the side of the tube and pull the swab back, you can squeeze a little in the tube this is soft plastic. Do this to allow as much liquid into the tube as possible.

6 - Fit the dropper on top of the extraction tube;

7 - Then drip at least 2 to 3 drops into the test cassette: Use the dropper to drip a few drops of the liquid from the tube onto the indicated spot on the test cassette. Avoid air bubbles

8 - Wait for the result: Place the test cassette on a flat surface and wait 10-15 minutes. If necessary, set a timer for your impatience.

Positive result?

1 - Consult a doctor: Contact your doctor or an STI clinic for further testing and/or treatment.

2 - Order a laboratory test: A quick test is often not enough for a doctor. Therefore, order a laboratory test. This is more reliable and often necessary before starting treatment.

3 - Inform sexual partners: Let your current and recent sexual partners know so they can also get tested and/or treated.

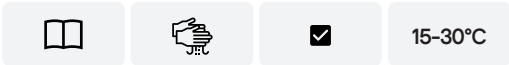
4 - Follow treatment instructions: Follow the prescribed treatment course completely and make sure your partner(s) do the same to prevent reinfection.

Negative result?

Stay alert: A negative test does not mean you are immune.

Regular testing: Consider regular STD testing if you are sexually active.

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Performance characteristics

1 - 1025 cases were detected simultaneously with the Encode Chlamydia trachomatis antigen test kit and the same kind of product by blind method. The results were as follows.

		Contrast segments		
		positive	negative	total
Encode	positive	201	34	235
	negative	15	775	790
total		216	809	1025
Positive coincidence rate		93.1%		
Negative coincidence rate		98.8%		

2 - 1000 cases were detected simultaneously with the Encode Neisseria Gonorrhea antigen test kit and the same kind of product by blind method. The results were as follows.

		Contrast segments		
		positive	negative	total
Encode	positive	219	6	225
	negative	10	765	775
total		229	771	1000
Positive coincidence rate		95.6%		
Negative coincidence rate		99.2%		

3 - Analytical Sensitivity:

	Analytical Sensitivity	Type	Source
Chlamydia	2x10 ³ IFU/ml	Inclusion body antigen	maxmed laboratories.inc
Gonorrhea	3x10 ⁴ CFU/ml	ATCC 19424	GDMCC

4 - Analytical Specificity: Compared with different kinds infection factors and the test result are negative

		Concentration	Source
Candida albicans	ATCC1023	6*10 ⁴ CFU/ml	GDMCC
Candida tropicali	ACCC20005	6*10 ⁴ CFU/ml	
Candida near smooth	ACCC20221	6*10 ⁴ CFU/ml	
Streptococcus fea cium	ATCC29212	6*10 ⁴ CFU/ml	
Proteus mirabilis	CMCC49005	6*10 ⁴ CFU/ml	
Staphylococcus fea cium	ATCC6538	6*10 ⁴ CFU/ml	
Escherichia coli	ATCC8739	6*10 ⁴ CFU/ml	
Pseudomonas aer uginosa	ATCC9027	6*10 ⁴ CFU/ml	
mycoplasma hominis	ATCC23114	1*10 ⁴ CCU/ml	Zhuhai Skin Disease Prevention and Control Institute
Ureaplasma urealyticum	ATCC2781	1*10 ⁴ CCU/ml	

5 - Interference: after testing, the following interferences have no effect on the results. 50 ul/ml whole blood, 5mg/ml mucoprotein, 50ul urine, 5mg/ml nystatin, 5mg/ml miconazole, 5mg/ml tinidazole, 5mg/ml metronidazole (gels), 50ul/ml Jieeryin (lotion), 50ul/ml Fuyinjie (lotion)

6 - Intra-batch Discrepancy: The test results are the same in one batch.

7 - Inter-batch Discrepancy: The test results are the same in different batch.

8 - Hook effect: No high dose hook effect was observed when tested with up to a concentration of 2.0x10⁸ ifu/ml chlamydia positive reference, and 3.0x10⁸ cfu/ml gonorrhea positive reference

Glossary of symbols

REF	Catalog number		Temperature limitation
	Consult instructions for use	LOT	Batch code
IVD	In vitro diagnostic medical device		Use by
	Manufacturer		Contains sufficient for <n> tests
	Do not reuse		Guard against damp
	Keep out of the direct sun	IC RP	Authorized representative in the European Community
CE	CE marked to IVD Medical Devices Directive 98/79/EC		