HIV-Syphilis Combinatie test

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HIV-Syphilis Combination test

Before you begin this test, read the instructions carefully, ensure that the contents of this test have been collected, and wash your hands before administering the test.

If any parts of the test are missing, administer a new test to ensure accurate results.


Kit contents:

- Test device
- Alcohol swab
- Pipette
- Buffer bottle
- Sterile lancet
- Instructions for use

Before you start the test

Please note: for safety and hygiene reasons, the lancet can only be activated once. This is done by placing the lancet against your fingertip and pressing it.


Once the lancet is activated, it can no longer be used, and you should use a new one for optimal test results.

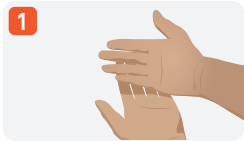
Before starting the test, clean your fingertip with the alcohol wipe.

Prick one of your fingertips with the lancet until a drop of blood is released.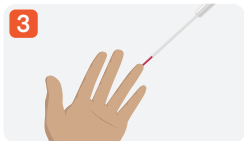

Collect 2 drops (80 µl) of your blood in the pipette.

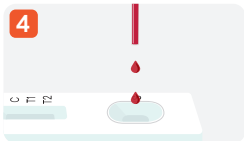Take the pipette and place 2 drops of blood in the sample compartment (letter S).

Add 1 drop from the buffer bottle to the sample compartment.

Read the result between 15 and 20 minutes after adding the drops.

Negative		HIV Positive		Syphilis Positive		HIV & Syphilis Positive		Invalid	
	C		C T1		C T1		C T1		C T2
	T2		T2		T2		T2		T2

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