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This kit includes:

- 1x Test urine tube with pipette (for urinalysis male)
- 1x Swab cotton swab in test tube (for vaginal/oral or anal examination)
- 1x A return box or envelope with a reply number sticker
- 1x Results card + sample number
- 1x Safety Bag
- 1x Results card + sample number

Note!

The front of the package of this test indicates which test to perform. Check carefully which instruction to follow when testing.

Instructions vaginal (female)

1 - You will need this code later to retrieve the results.

2 - Spread the labia and hold the cotton swab. Insert the swab 5 cm into the vagina. Stroke along the vaginal wall for 30 seconds.

3 - After you are done, place the cotton swab back into the test tube without touching any other surfaces.

4 - Turn the tube tightly closed. Put it in the Safety Bag and tape it tightly shut. Put the Safety Bag in the return box and fold/tape it closed.

Instructions urine (man)


1 - You will need this code later to retrieve the results.

2 - Preferably take morning urine, or wait at least two hours after your last urination before starting the collection. Take a clean cup (e.g. disposable cup) and collect the first drops/jet of your urine (max. 30 ml).

3 - Open the tube. Use the pipette to transfer 1.5 ml of urine into the tube.

4 - Turn the tube tightly closed. Put it in the Safety Bag and tape it closed. Put the Safety Bag in the return box and fold/tape it closed.

Oral instructions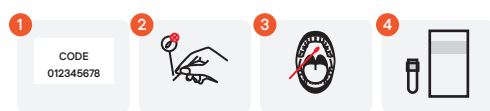


1 - You will need this code later to retrieve the results.

2 - Remove the swab from the package. Make sure it does not come into contact with anything.

3 - Insert the cotton swab into the back of your throat and gently rotate it and rub firmly along the left and right palate arch. Note that this may cause a gagging sensation. If this happens, take a break and then continue until you have rubbed at least 6 times.

4 - After you are done, place the cotton swab back into the test tube without touching other surfaces such as cheek or tongue.

Anal instructions

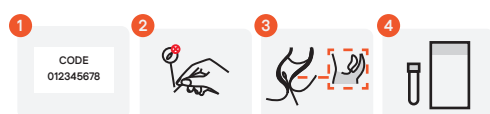1 - You will need this code later to retrieve the results.

2 - Remove the swab from the package. Make sure it does not come into contact with anything.

3 - Carefully insert the swab 3-5 cm into the anus and rotate it for 5-10 seconds.

4 - Place the swab in the test tube without touching other surfaces. Now twist the tube tightly. Put it in the Safety Bag and tape it closed.

Curious about the results?

The laboratory usually analyzes your sample immediately on the day of receipt or the following business day. At the end of each business day (Monday through Friday), results are uploaded after 6 p.m. and are available for retrieval through the website. For test results go to

<https://uitslag.thuistest.shop>

Scan QR code for a digital version

Ready for shipping?

After you seal the Safety Bag, carefully place it in the return box. Send the return box or return envelope with reply number free of charge to the laboratory, by putting it in the mailbox before the collection time. Usually this is on a weekday before 17:00 in an orange street mailbox of PostNL or red mailbox of Bpost (Belgium). The time when the mailbox is emptied may vary from one mailbox to another. Check the PostNL location guide for the emptying times or drop off your package at the post office.

Is it colder than 0 degrees or warmer than 30 degrees outside? If so, always drop off at a post office.

Tip: take a picture of your code card with sample number so you don't lose it to see the result!